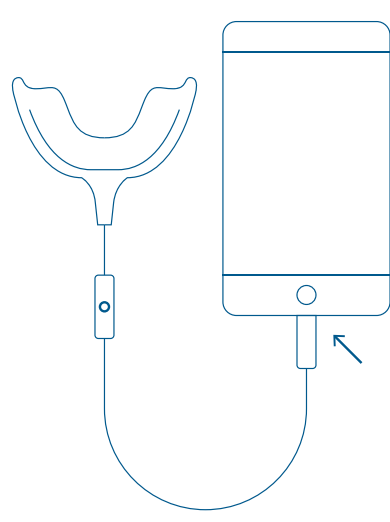
WHITE U

SMILE WHITENING KIT

ISTRUZIONI D'USO HOW TO USE

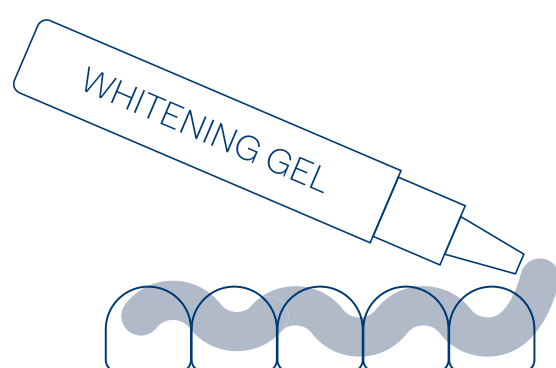
1

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Connect the LED light cable to your Smartphone.

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
Open the whitening pen, apply a layer of gel to teeth, approximately 1 mm thick.

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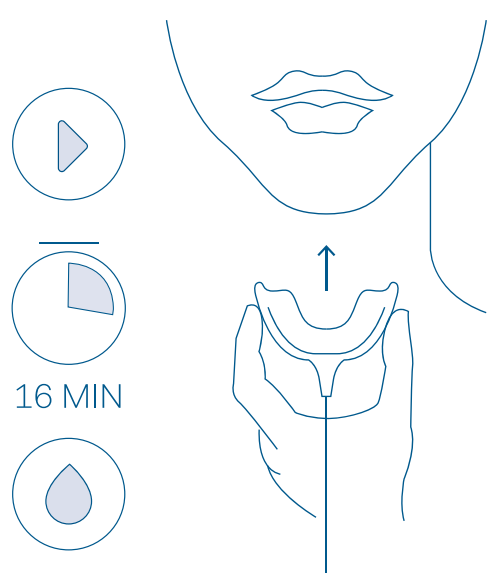
Insert the LED mouth piece, press the start button.

The light activates the gel and turns off automatically after 16 minutes.

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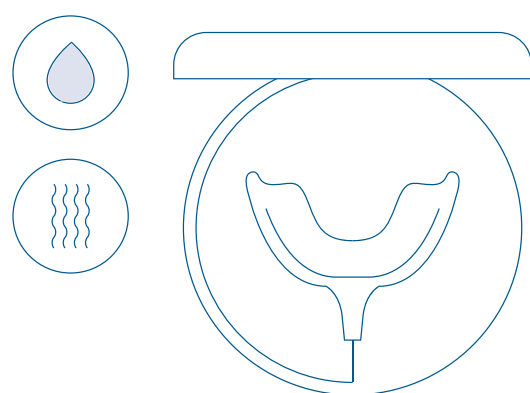
After use, rinse your mouth and the LED mouth piece with water, dry the mouthpiece and store it in its protective case.

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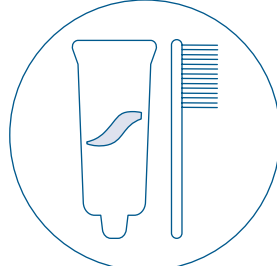
Clean teeth by brushing with WHITE U toothpaste before treatment

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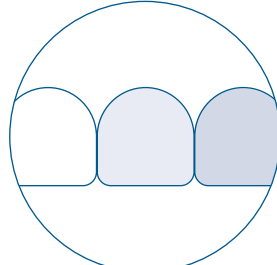
Avoid the intake of highly pigmented foods, tobacco and coloured beverages such as wine, coffee, tea before and after application.

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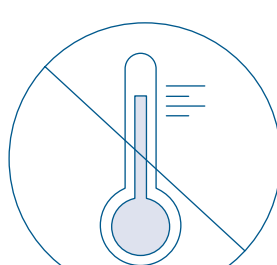Use the smile shade guide to monitor your results.

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Do not clean the LED mouthpiece with boiling water.

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For best results we recommend a daily use of WHITE U SMILE WHITENING KIT.

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Keep out of reach of children. Contains easily swallowed parts. Product for cosmetic dental use only, to be used only by adults. Not intended for use in children under the age of 12. All of cosmetic whitening products are not a substitute for regular dental care. Please seek medical advice prior to using in case of pregnancy, breastfeeding, allergies to resins/chemicals, pathologies and/or recent oral procedures. Do not swallow.

APPARECCHIATURE ELETTRICHE ED ELETTRONICHE
EQUIPMENT ELECTRIC AND ELECTRONICS

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© Copyright. All rights reserved